Cat. No 1468

GET READY!

GB	Breast shells - milk collectors	3
PL	Muszle laktacyjne - kolektory mleka	4
BG	Лактационни черупки - колектори за мляко	5
CZ	Laktační mušle - sběrače mateřského mléka	6
DE	Milchauffangschalen	7
EE	Rinnakaitsed - piimakogujad	8
IT	Conchiglie raccogli latte per allattamento	9
RO	Cochilii de lactație - colectoare de lapte	10
SK	Laktačné mušle - zbierače mlieka	11

babyono





MANUAL

I

II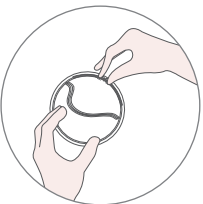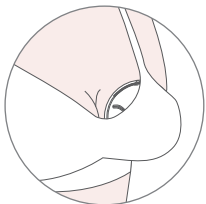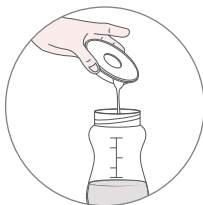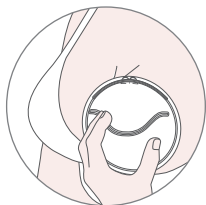

Breast shells - milk collectors

Operating manual

Breast shells – milk collectors are user-friendly and support breastfeeding. Their role is to collect milk coming out of the breast while breastfeeding or expressing milk. The set includes two collectors and a handy storage case. The covers also protect irritated nipples against chafing.

Components of the product (fig. I)

- A_silicone membrane
- B_cover
- C_air hole
- D_silicone plug
- E_nipple hole
- F_storage case

How to use? (fig. II)

Shells with plugged air hole – use while breastfeeding or using a breast pump

1. Put a silicone membrane (B) on the cover (C). Put the silicone plug (E) on the air hole (D). Make sure all components of the collector have been properly installed.
2. Insert the collector in the bra, make sure the hole (F) in the silicone membrane fits the nipple.
3. When milk is already in the container, pour it into the bottle/storage bag and leave in the fridge.
4. Wash the collector under running water and leave in the airy place to let it dry.

Shells with open air hole – use between breastfeeding sessions, when your nipples are already sore and you want to make sure your clothes do not make them further irritated.

1. Put the silicone membrane (B) on the cover (C). Leave the air hole (D) open to assure air circulation. To do so, pull the silicone plug (E) out. Make sure all components of the collector have been properly installed.
2. Insert the collector in the bra, make sure the air hole (D) is facing upwards. The hole (F) in the silicone membrane must fit the nipple.
3. When milk is already in the container, pour it out. The milk in the shell must not be served to the baby. You can pour it into the bathtub, for instance.
4. Wash the collector under running water and leave in the airy place to let it dry.

WARNING!

1. The product is not a toy. Keep it out of reach of children.
2. Use the product as per intended use.
3. Only clean breast shells must be used.
4. Store shells in a dry place, avoid sunlight.
5. If you can notice any signs of damage, replace breast shells with new equivalents.

Cleaning

IMPORTANT: Before the first use and after every use, wash and disinfect breast shells and silicone inserts.

For cleaning, use soft brushes or sponges. Hard and/or sharp materials may scratch the product. Disinfect breast shells in the steam steriliser, or boil them in hot water for about 4 minutes.

The silicone plug (E) has a special hole, if you switch one tip into another you can turn the plug into the holder that you can use to hang the product for drying.

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